



STEP 7

CUSTOMIZE THE MEGASUS HORSERUNNERS


5. If needed, insert the stabilizer.

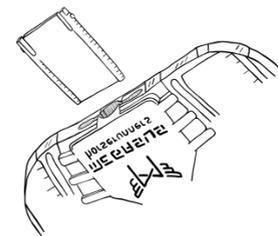

6. With the jigsaw, cut 4mm away from where you traced the hoof on the Megasus Horseshoers (or on the dashed line).

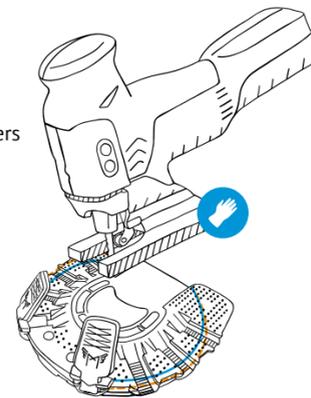


7. Use the angle sander to smooth any sharp edges.

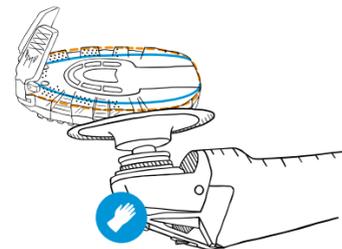


STEP 8

CUSTOMIZE THE MEGASUS HORSERUNNERS


8. In the back region of the Megasus Horseshoers or the heel, sand the back end of the Megasus at an inward angle. The tracing should remain, but underneath, the sole should be shorter. That way, it will be more difficult for the horse to step on the back of the Megasus Horseshoers and pull them off.

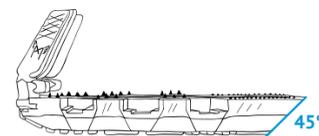


9. Check to see if the slots for the MEGA-GLUE Side-Clips were cut. If yes, then the back end of the clips will also need to be adjusted to fit the slots. Use the nippers and cut the end of the MEGA-GLUE Side-Clips.

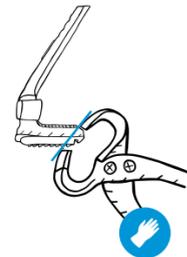

10. Now insert all the MEGA-GLUE Side-Clips until the outer wings line up with the edge of sole.

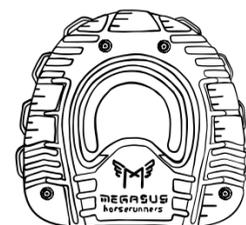

- STUDS OR SPIKE**
Optional step: insert the studs or spikes.

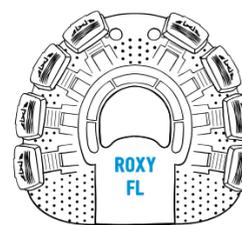


- LABEL YOUR MEGASUS**
Name of the horse and the hoof (right, left, front, back)

STEP 9


APPLY THE MEGA-GLUE SIDE-CLIPS

1. Clean the hoof with the isopropyl alcohol and allow to air dry.

WARNING: from this point on, you should not touch the hoof wall!

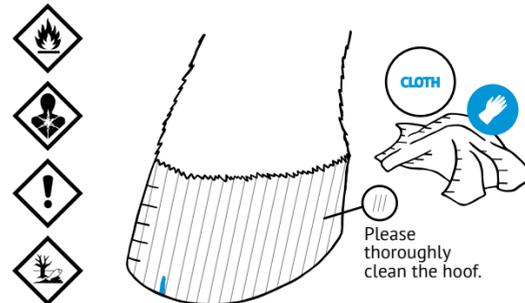

2. Lift the hoof up and the push the Megasus Horseshoers on from back to front. Make sure the middle marking on the toe and middle marking on the Megasus align.

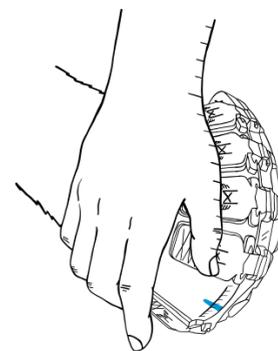


STEP 10


APPLY THE MEGA-GLUE SIDE-CLIPS

3. Drive in the 6-10 MEGA-GLUE Side-Clips using the Mega-HAMMER or a hammer and a drift by striking the orange stem of the MEGA-GLUE Side-Clips until they are so far into the slot that the blue wings lie firmly on the hoof.

It is recommended to alternate driving the MEGA-GLUE Side-Clips in to help avoid any unevenness.

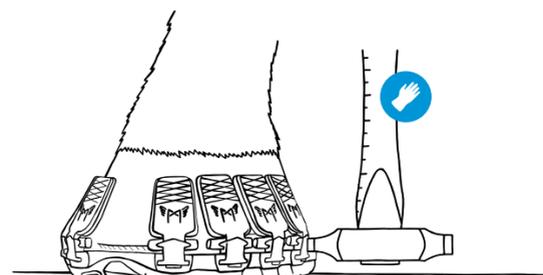

4. Apply all the MEGA-GLUE Side-Clips firmly to the hoof and wrap an elastic band or use a plastic wrap roller to wrap the perimeter of the hoof and Side-Clips to keep them in place during gluing. The overhead area of the MEGA-GLUE Side-Clips should be on the hoof, but not bound by the elastic band or plastic. This way, you will be able to fill the MEGA-GLUE Side-Clips with glue. If using plastic wrap and it's too high up on the hoof, simply push the plastic wrap down with your fingers.

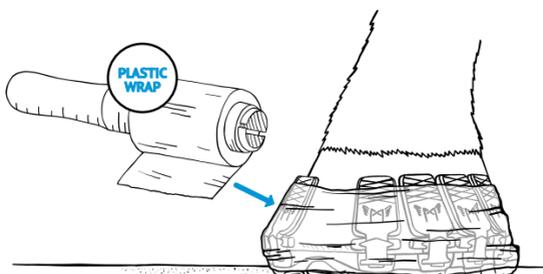

STEP 11

FASTEN THE HORSERUNNERS


1. Insert the MEGA-GLUE 2C-GLUE into a manual cartridge glue gun. Open the MEGA-GLUE cartridge and attach the mixing tip. To test the glue, squeeze out the first portion until the two glues have properly mixed. Now the MEGA-GLUE Side-Clips will be quickly filled with the Mega-Glue.

Because the MEGA-GLUE Side-Clips, when being filled with glue, move away from the hoof, stick the mixing tip to the bottom end of the Side-Clips and carefully and evenly inject the glue into the Side-Clip, gradually filling in a zigzag motion until the Side-Clip is full. Immediately begin filling the next MEGA-GLUE Side-Clip.

WARNING: The MEGA-GLUE hardens quickly, only needing about 30-45 seconds to dry. You may need to pre-warm your MEGA-GLUE cartridge: the warmer the glue, the more fluid it will be and will harden quicker. In the cold, the glue must be pre-warmed in a heating pocket or something similar.




WARNING: If the MEGA-Glue runs out on the side or up top, immediately wipe it away with your finger - please wear hand gloves!

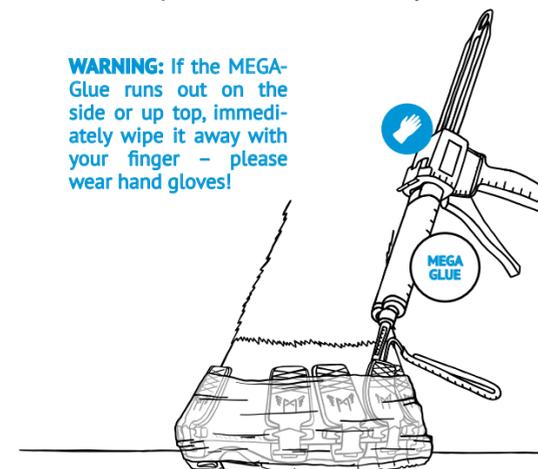

STEP 12

FASTEN THE HORSERUNNERS


2. Now place the hoof on the hoof stand to make the final touches to the sole. Sand down the orange overhang on the Side-Clips or any sharp edges with the hoof rasp. Make sure not to damage the upper part of the Side-Clips when rasping.

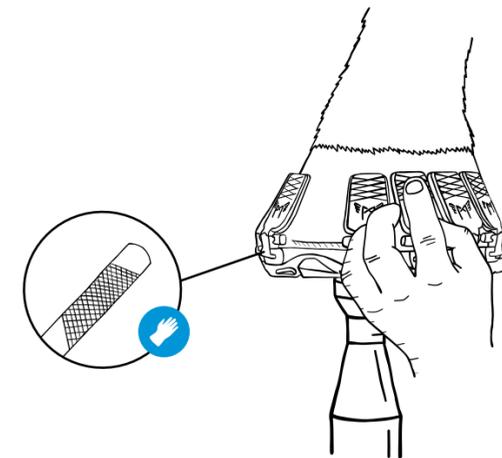

INFO For the next hoof, please use a new mixing tip.



MOUNTING INSTRUCTIONS

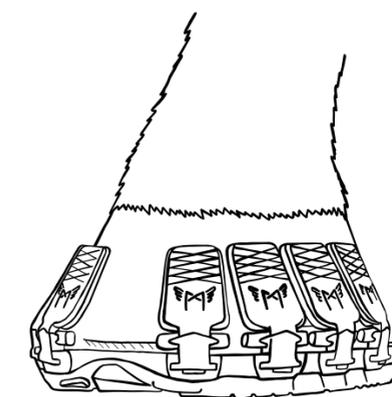

MEGA-GLUE SYSTEM

Please view our mounting videos!
www.horseshoers.com



STEP 1

CHECKING THE CONTENT

The set of 2 contains 2 MEGASUS HORSERUNNERS and all the necessary components for the MEGA-GLUE SYSTEM.




36 MEGA-GLUE SIDE-CLIPS



2 STABILIZER



2 MEGASUS HORSERUNNERS

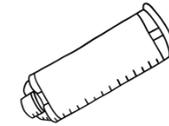

1 MEGA-GLUE 2 COMPONENT GLUE



8 MIXING TIPS

2X50ML MEGA-GLUE UP TO SIZE 160


3X50ML MEGA-GLUE FOR SIZES 180 & 200

PLEASE NOTE

- 1. PROTECT THE ENVIRONMENT!**
Use a trash bin, broom or vacuum cleaner.
- 2. WHAT'S BEEN CUT AWAY CAN'T BE REAPPLIED!**
It is better to cut less than too much. What's been cut away can't be reapplied.
- 3. DRY, CLEAN AND GREASE FREE!**
The hooves must be dry and grease free (for a minimum of 14 days). No hoof grease or oil, finger prints, fly spray or anything similar
- 4. YOUR ABILITIES!**
To customize the Megasus Horserunners, you will need to use a variety of machines/tools to do so.

STEP 2


CHECKING THE CONTENT

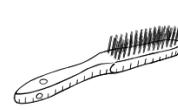

1. WIRE BRUSH




2. RASP

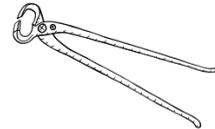

3. NIPPERS




4. PERMANENT MARKER

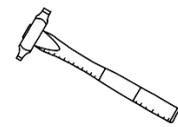

5. HAMMER WITH A DRIFT OR SPECIAL PURPOSED MEGA-HAMMER




6. HOOF BUFFER OR SANDPAPER

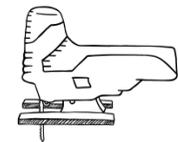

7. JIGSAW



8. ANGLE GRINDER

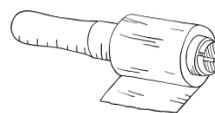

9. PLASTIC WRAP




10. HOT AIR GUN

STEP 3

PREPARE YOUR SURROUNDINGS

1. COMPACT SURFACE
3. CLEAN HOOF

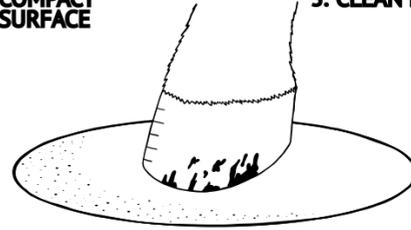

2. AS CLEAN AS POSSIBLE
4. DRY HOOF

TEMPERATURE 15 - 25 °C | 59 - 77 °F

HAND GLOVES

HOT AIR GUN For temperatures below 15 °C | 59 °F

MOUNTING IS NOT POSSIBLE

GREASY HOOVES



wait at least 14 days and degrease the hooves (Isopropylalcohol)

WET/MOIST HOOVES




allow the hooves to sit in a dry area and use a hot air dryer (intermittently let the hoof cool and brush)


STEP 4

PREPARE THE HOOF


1. Clean the hooves with the wire brush.

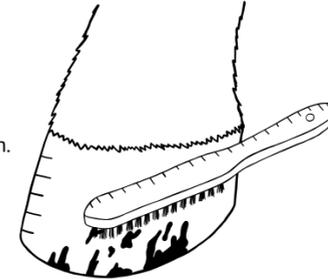

2. Abrade the hoof wall with the hoof buffer or with sandpaper.

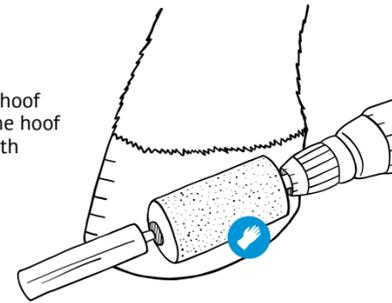

3. Rasp the hoof edges at a right angle, 7 millimeters to the ground.

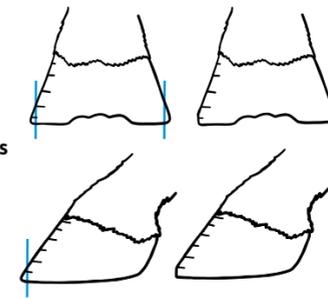

STEP 5


CUSTOMIZE THE MEGASUS HORSERUNNERS

HOOVES MUST BE CLEAN, DRY, AND DECREASED*!


*FOR A MINIMUM OF 14 DAYS

1. Drive both front 2 MEGA-GLUE Side-Clips into the adjustment slots on the toe with the hammer or MEGA-HAMMER.

CAUTION: Only insert them as far as the wings of the MEGA-GLUE Side-Clips are in line with the outer edge of the sole.

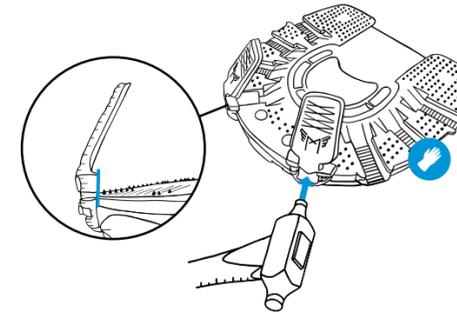

2. Lift the hoof up and with a permanent marker, mark on the toe where the center of the hoof is and mark the middle point on the toe of the Megasus sole.

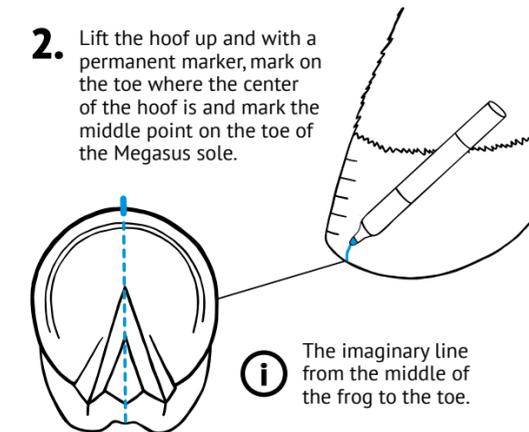


The imaginary line from the middle of the frog to the toe.

STEP 6


CUSTOMIZE THE MEGASUS HORSERUNNERS

3. Lift the hoof up and place the Megasus Horserunners with the front MEGA-GLUE Side-Clips on the toe so that the middle marking on the toe matches the middle marking on the Megasus sole, between the two MEGA-GLUE Side-Clips.

Make sure to also keep the rear of the hoof centered on the Megasus and use the permanent marker to trace the outline of the horse's hoof on the Megasus sole, also marking the length in the back of the Megasus so that you know how much the Megasus needs to be shortened.

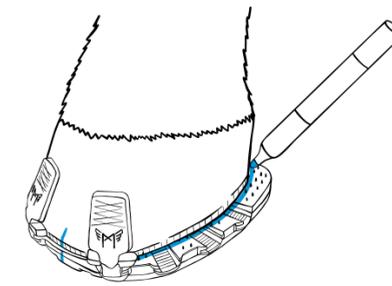

4. Now that you have the hoof shape traced on the Megasus Horserunners, draw a separate dashed line around the first marking with permanent marker, about 4 mm outside. The dashed line is where you will cut the Megasus Horserunners with the jigsaw.

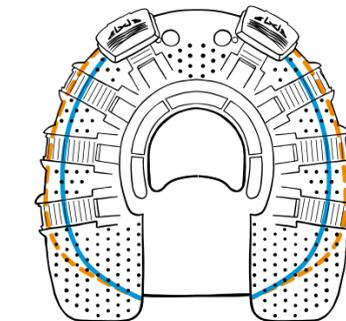

CUTTING LINE (4 mm outside hoof outline)

HOOF OUTLINE



KEEP OUT OF REACH OF CHILDREN.

Highly flammable liquid and vapor. Causes eye irritation. May cause an allergic skin reaction. May be fatal if swallowed and enters airways. May cause drowsiness or dizziness. May damage fertility or the unborn child. Suspected of causing cancer. Causes damage to organs: sensory organs. Causes damage to organs through prolonged or repeated exposure: nervous system. May cause damage to organs through prolonged or repeated exposure: sensory organs.

Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Keep away from heat/sparks/open flames/hot surfaces. - No smoking. Ground/bond container and receiving equipment. Use only non-sparking tools. Take precautionary measures against static discharge. Keep container tightly closed. Use explosion-proof electrical/ventilating/lighting equipment. Do not breathe dust/fume/gas/mist/vapors/spray. Use only outdoors or in a well-ventilated area. Wear protective gloves and eye/face protection. Do not eat, drink or smoke when using this product. Wash thoroughly after handling.